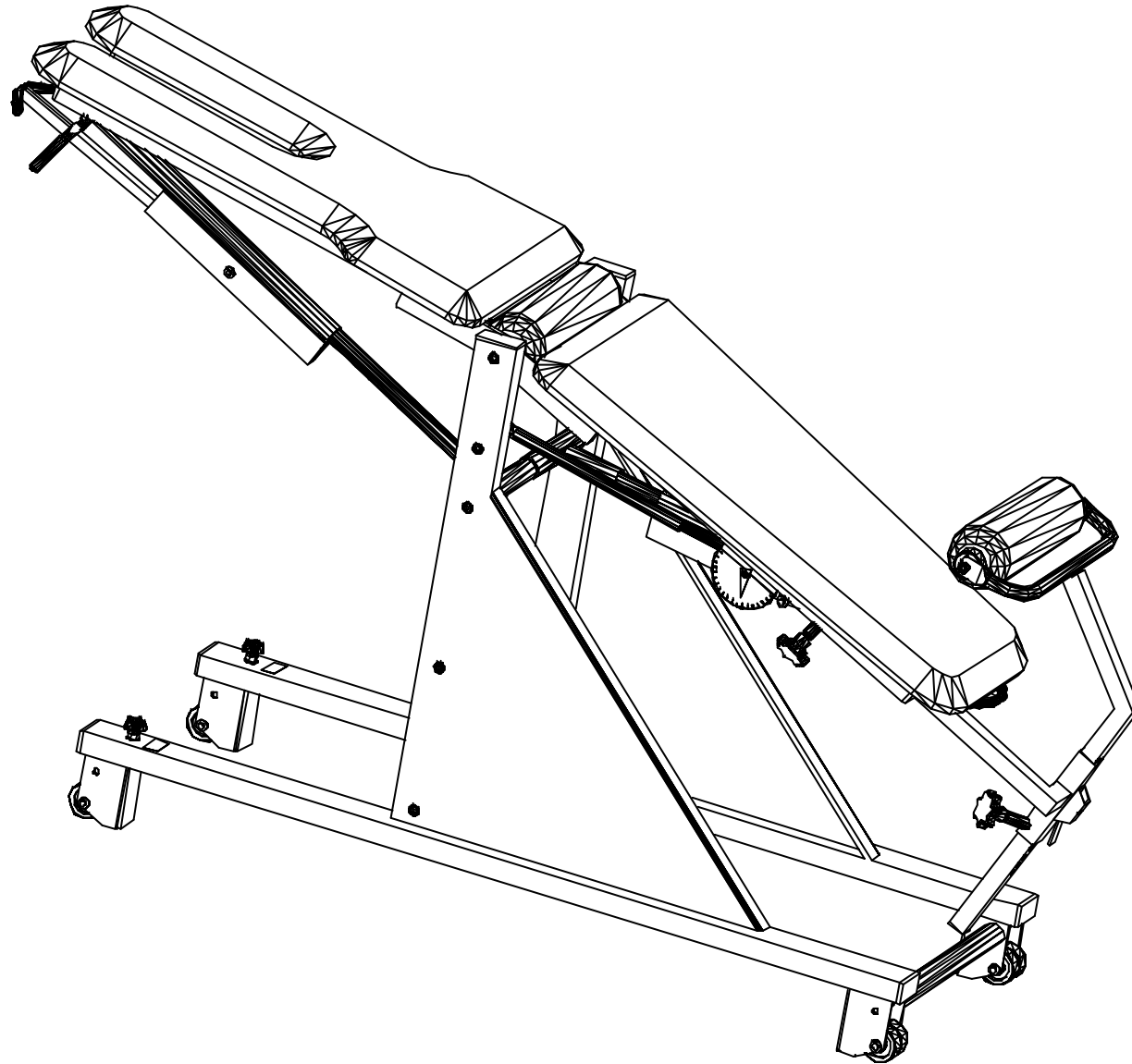
FOLLO DIEM ADJUSTABLE

FOLLO DIEM ADJUSTABLE

We congratulate you on choosing a Follo Diem product. This instruction information makes it able for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.
Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:
COLOUR:
SERIAL NO.:


FUNCTION TEST:	
<input type="checkbox"/> Frame (screws, plastic parts, surface, welding)	<input type="checkbox"/> Labels
<input type="checkbox"/> Cushions (colour, screws, surface)	<input type="checkbox"/> Adjustments

ASSEMBLED (date/sign.):	
-------------------------	--

END CONTROLLED (date/sign.):	
------------------------------	--

PACKING CONTROL: (date/sign.):	
--------------------------------	--

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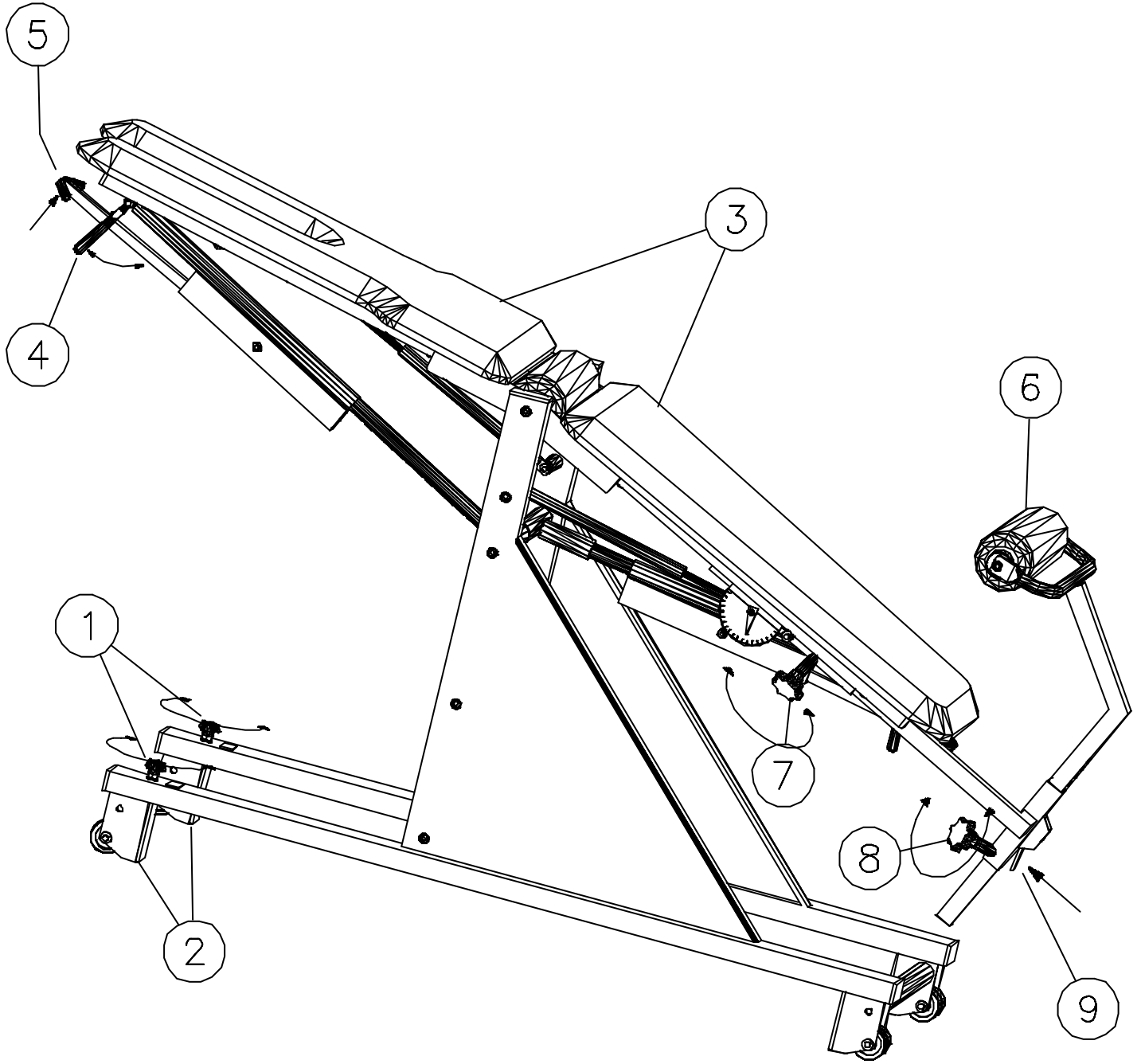FOLLO DIEM ADJUSTABLE

We congratulate you on choosing a Follo Diem product. This manual enables you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use. Contents:

- ✓ Unpacking/installation
- ✓ How to use
- ✓ Technical specifications
- ✓ Safety information
- ✓ Maintenance instructions
- ✓ Spare parts list

UNPACKING/INSTALLATION

1. Take care when removing packing materials. **DO NOT** use a knife or other sharp instrument to avoid damaging the contents.
2. Inspect the bench for transport damage.

SAFETY INSTRUCTIONS

1. Study and learn the operation of the handles and switches.
2. Take care to avoid pinching when making adjustments.
3. Do not overload the bench.

HOW TO USE

1. Brake and levelling adjustment: → Adjustment screws (1) are provided on two of the bench legs/wheels. Turn the adjusting screws so that the supporting blocks lift wheels just clear of the floor (2-5mm). Adjust so that the bench rests evenly on the floor.
2. Adjustment of cushion: → Both cushions are adjusting in the same manner. Move the locking handle (4) rearwards to disengage. Lift the arm (5) and adjust the cushion to the required angle/height. Release the arm (5) making sure it is properly engaged. Lock the arm in position (5) with locking handle (4).
3. Adjustment of foot cushion (6) outwards/inwards: → Loosen the locking screw (7) beneath and to one side of the cushion to which the foot cushion is attached. Adjust the foot cushion to the required position and tighten the locking screw (7).
4. Adjustment of foot cushion (6) up/down: → Loosen the locking screw (8) for height adjustment. Depress the spring of the locking device (9) to release the lock and adjust the foot cushion to the required height. Make sure that the locking device (9) is properly engaged. Tighten the locking screw (8).

MAINTENANCE INSTRUCTIONS

Check the following at regular intervals (every third month):

1. Make sure that the cushion and foot cushions are in good condition without visible damage.
2. Make sure that all wheels run smoothly.
3. Check that the locking handles/locking screws for cushion adjustment are functioning properly.

To maintain the appearance of PVC Coated Fabrics they should be cleaned regularly to remove fatty substances that may reduce their service life. A soft cloth and warm soapy water should be used to clean the surface of the material. Mild soap or soap flakes are recommended and residual soap should be removed using cold water. The surface should be scrubbed with a soft brush. Solvent cleaners, wax polishes strong detergents and patent chemical cleaners should not be used. The use of these substances is likely to harm the coat and repeated use can result in the removal of elasticity from the PVC coating, which will result in hardening and subsequent cracking of the material's surface. PVC Coated Fabrics are resistant to most mild acids, alkalis and household stains. The vinyl may absorb some substances such as ballpoint pen ink, lipstick, newsprint and food, and this may cause permanent staining. This can often be minimised by immediate cleaning with damp, soap cloth or sponge. Never place hot or cold items directly on PVC Coated Fabrics.

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FOLLO FUTURA A/S