

SIGN IN MYSPM GUIDE


STEP 1

SIGN IN OR LOG IN

To access **MySPM** you need **credentials**.

From this [LINK](#) you can register as an administrator of your facility.

If you have already registered, **log in** to your **admin area**.

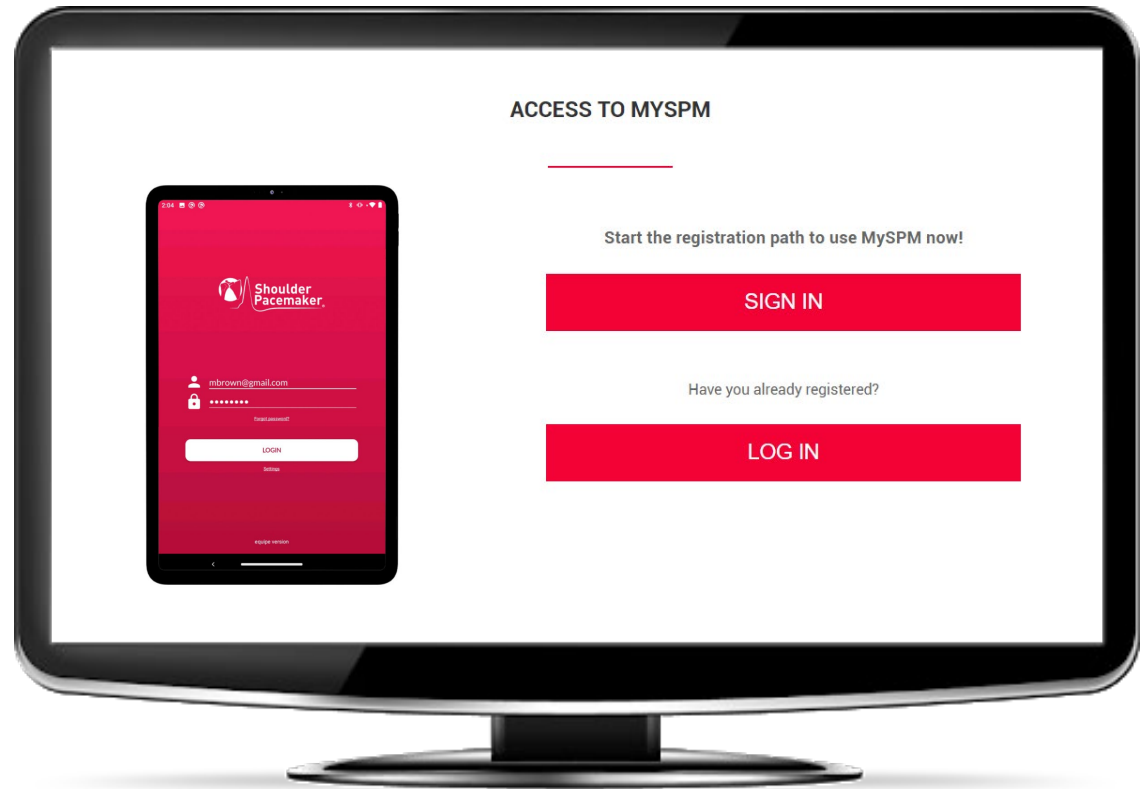


STEP 2

REGISTER SHOULDER PACEMAKER

Enter the **serial number** and the **unique code** of your device.

They are on the 2nd page of the supplied user manual.

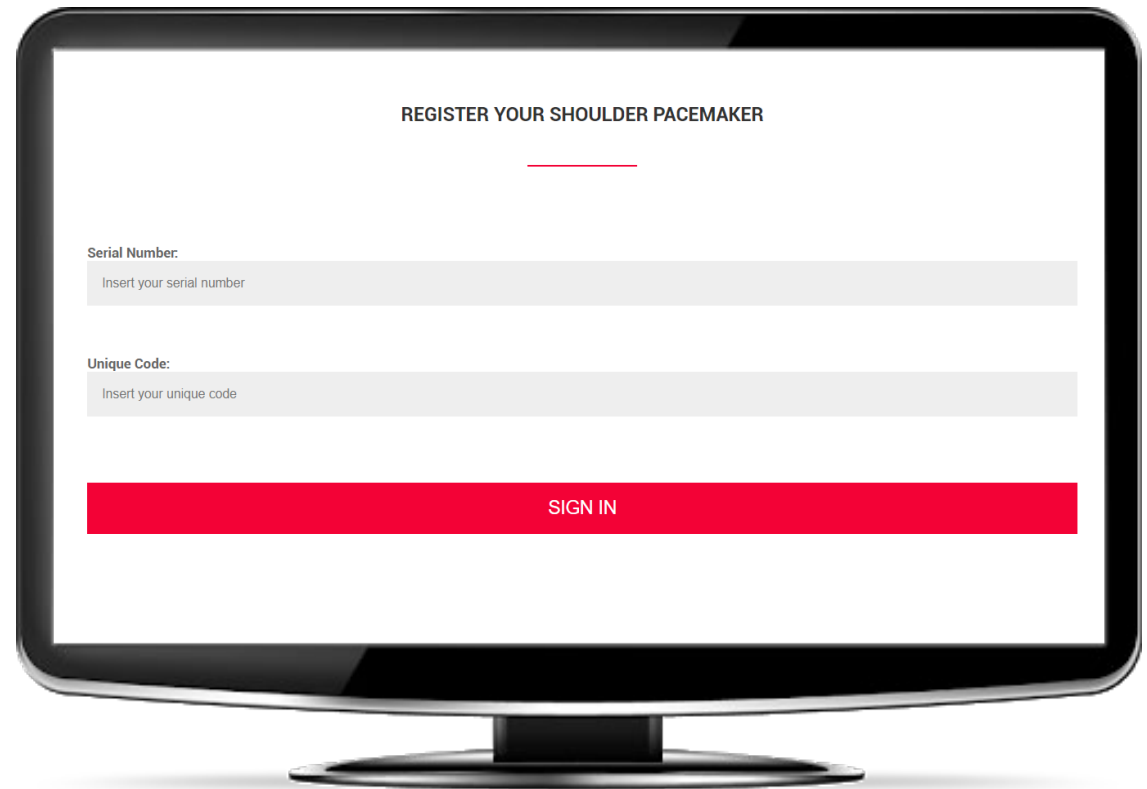

REGISTER YOUR SHOULDER PACEMAKER

Serial Number:
Insert your serial number

Unique Code:
Insert your unique code

SIGN IN

STEP 3

ENTER YOUR DATA


Enter your:

- **name**
- **surname**
- **email**
- **password**

WARNING

Make sure you enter a valid email

NB: Email and password will be your login credentials to MySPM app

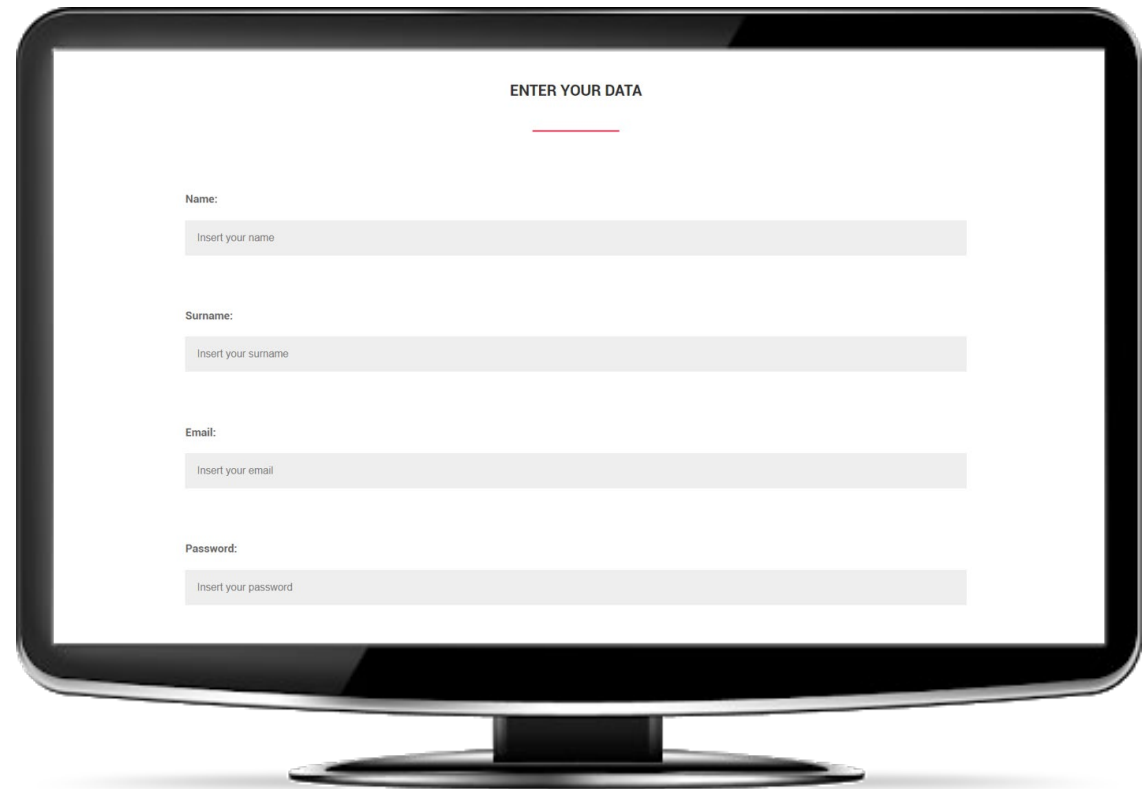

The image shows a computer monitor displaying a registration form. The form is titled "ENTER YOUR DATA" and contains four input fields, each with a label and a placeholder text:

- Name:** Insert your name
- Surname:** Insert your surname
- Email:** Insert your email
- Password:** Insert your password


STEP 4

REGISTER YOUR FACILITY

Enter the **name** and the **city** of your facility.

After pressing the *SIGN IN* button, an **email** will be sent to the address entered above.

To access the next step, you must click on the link in the email.

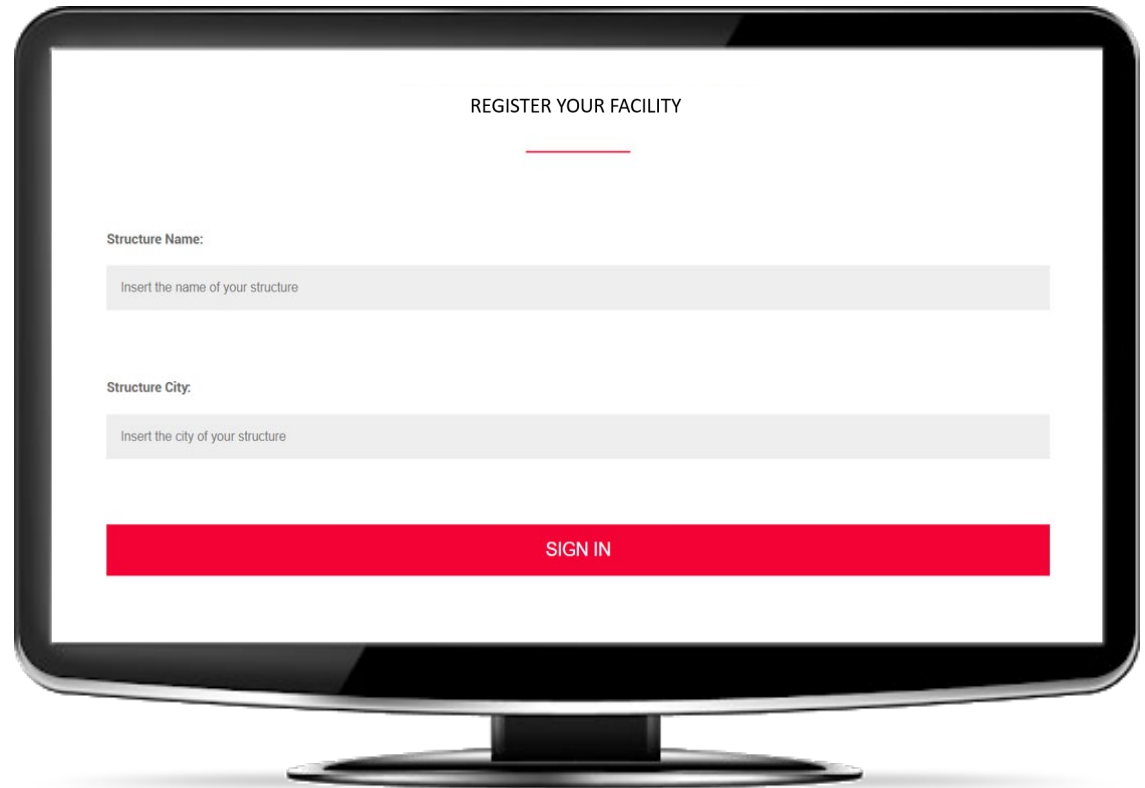

REGISTER YOUR FACILITY

Structure Name:
Insert the name of your structure

Structure City:
Insert the city of your structure

SIGN IN

STEP 5a

ADMINISTRATOR AREA

In the Administrator area you can see all your devices.


If you have more than one device, you can manage them with:

- ***DISTINCT FACILITIES***

Sign in as a new user repeating the registration procedure from Step 1 with the new couple of serial number and unique code. You need a different email address for each new facility account. It will not be possible to share patient data and treatment status between different facilities.

- ***UNIQUE SHARED FACILITY***

Log in with your current credentials and ADD other devices from your Administrator area. The information associated with the facility (patient data, treatment status, statistics, etc) will be shared within all the staff members.

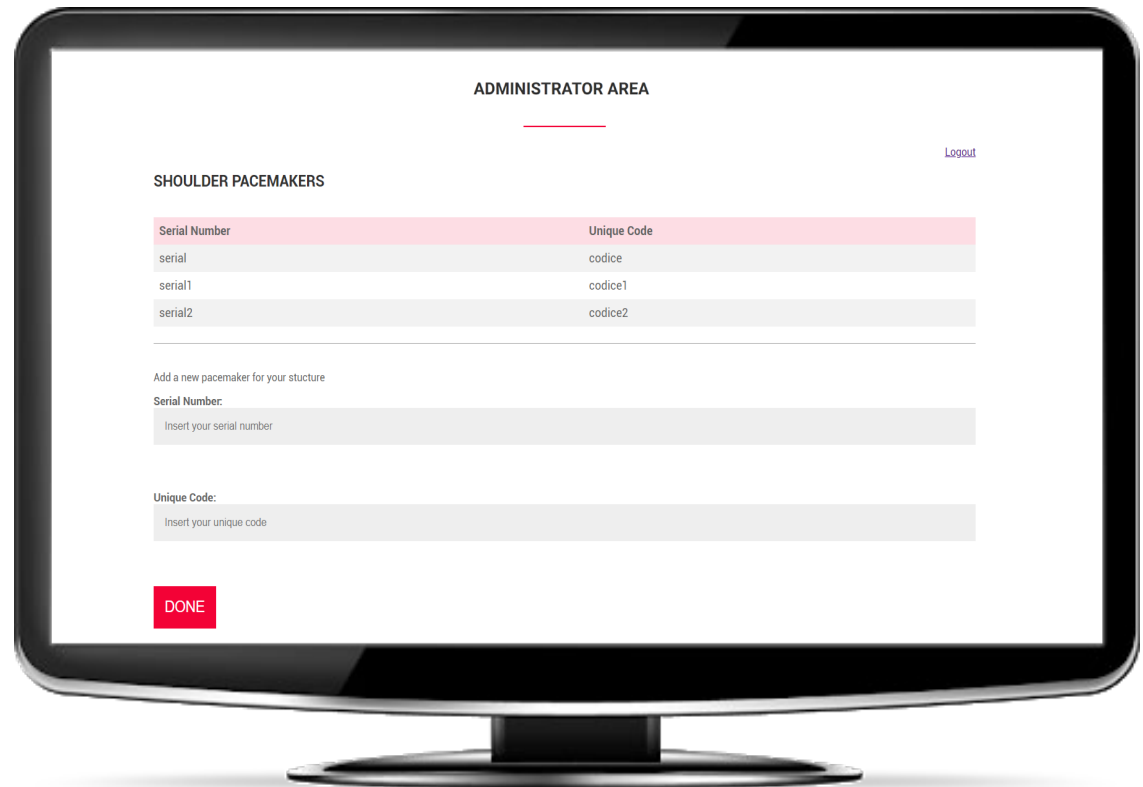

STEP 5b

ADMINISTRATOR AREA

In the Administrator area you can see all the people of your staff.

You can **add other** people.

WARNING

The people of your staff will be able to access the app and see the list of all patients registered in your facility.

