FOLLO STABILIS ARM TRAINER

We congratulate you on choosing a Follo Stabilis product. This instruction information makes it possible for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.
Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

ORDER NO.:	_____
COLOUR:	_____
SERIAL NO.:	_____

FUNCTION TEST:	<input type="checkbox"/> Surface: No damages
<input type="checkbox"/> Bottom frame: Wheels, Screws, plastic parts	<input type="checkbox"/> Labels: Serial number
<input type="checkbox"/> Crank: Securing the pin hole	
<input type="checkbox"/> Top frame: Surface, screws, plastic parts	ENCLOSED:
<input type="checkbox"/> Glide tube: Welding, surface, Silicon grease	<input type="checkbox"/> Instructions
<input type="checkbox"/> Adjusting handle	<input type="checkbox"/> Screws

ASSEMBLED (date/sign.):	_____
-------------------------	-------

END CONTROLLED (date/sign.):	_____
------------------------------	-------

PACKING CONTROL: (date/sign.):	_____
--------------------------------	-------

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The purpose of the training

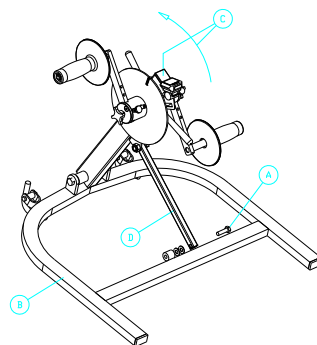
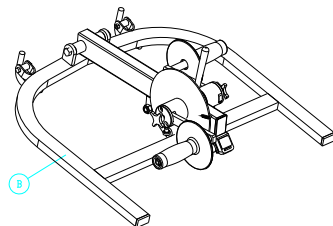
The arm training machine is designed to improve circulation. It can be used standing on a table or on the floor or it can be attached to wall bars. Select a resistance that the patient can keep up for at least 15 minutes. Rotate using even movements. Movements should be extended as far as possible. Read off time and distance on the counter.

Safety information

- Make sure that nothing can be pinched or jammed and that the handles rotate easily and are properly secured.
- Always test the machine's functions yourself and allow the patient to test all functions under supervision before handing out the product for self-training.

Unpacking/assembly



- Take care when removing the machine from its packaging. Do **NOT** use a knife or other sharp instrument as this may damage the machine. Inspect the machine for any transport damage.
- Place the machine on the floor as shown in the drawing. Remove screw (A). Stand on the frame (B) and lift up the main beam (C) until the angled supporting rod (D) lines up with its attachment point. Replace screw (A). Place two washers between the supporting rod and the attachment point. Do not use a washer on the outside of the supporting rod (see drawing).

Guidelines for use


1. The arm trainer machine is designed for use standing on a table or attached to wall bars. The person using the machine may sit or stand in front of the machine. Make sure the machine is properly attached to the wall bars or table and is steady.

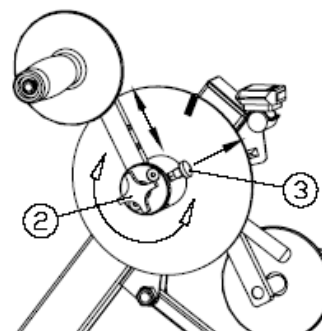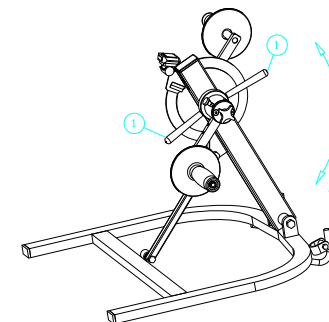
2. Adjusting resistance: → Adjust resistance using the levers (1). Rotating the levers in a clockwise direction increases resistance, and loosening anti-clockwise reduces resistance.

3. Adjusting the handles: The crank throw of the handles may be adjusted by first loosening the knob (2). Pull out the lockpin (3) and push or pull the handle to the required position. Make sure that the lockpin (3) lines up with a hole and replace the lockpin. Tighten the knob (2).

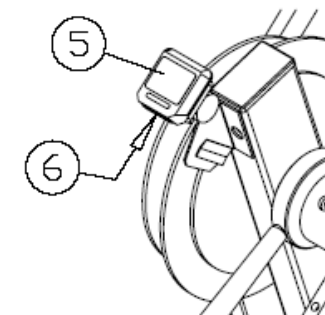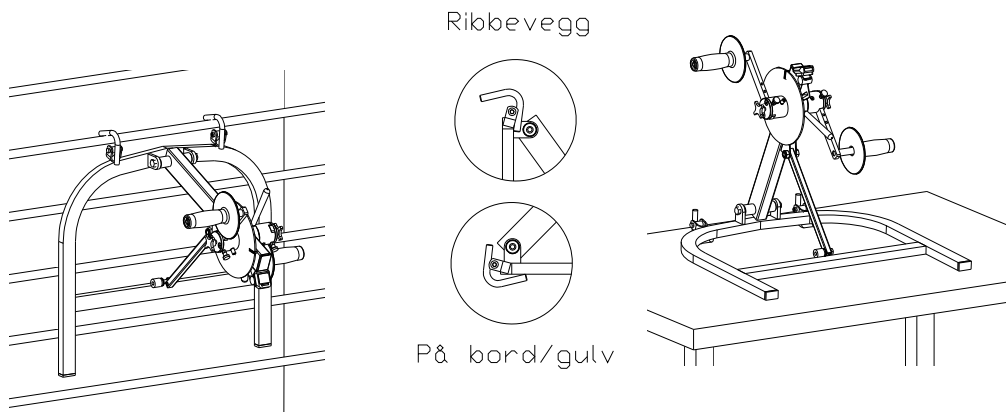4. Counter: The counter (5) is a bicycle computer. It registers when training starts, and is switched on automatically (at the same function as when it was last used). Use the button (6) on the bicycle computer to switch between the different functions: Distance, time, speed, clock and auto (changes regularly between all functions). Holding down button (6) will clear time and distance (not the clock). The computer will switch off automatically when not in use for a period. Read more about the bicycle computer's functions in the separate instructions for use.

5. Use on wall bars or on table/floor: → The machine is designed for use on wall bars, on a table or on the floor. As shown in the drawing above the forward attachments on the frame are plastic coated hooks which are used either to hang the machine on wall bars or as a feet when using the machine on table or floor.

Maintenance

Inspect the machine regularly for damage, play or missing parts. In the case of damage or other deficiencies, the machine must not be used until it has been repaired.

Trouble shooting

- **Counter / computer:**
If the counter stops working (empty display), check the battery first. If the counter is working, but does not count, check that the black transmitter on the largest nickel plated disc (No. 28 on the parts drawing) is in position. Check also that the opening between the black transmitter and the sensor (immediately below the computer) is correct when they pass each other. The opening must be between 2 and 4 millimetres (maximum 5 millimetres). Adjust the opening by carefully bending the baseplate to which the sensor is attached. See also the folder that accompanied the computer.
- **Brake resistance:**
If no brake resistance is felt or the resistance is not functioning properly, the handle may have been adjusted too far in the wrong direction (several turns). Turn the handle clockwise until heavy resistance is felt. Then adjust to the required resistance. If resistance is uneven, inspect the brake lining (No. 29 on the parts drawing). If the brake lining has loosened, it must be replaced.

If these guidelines fail to rectify the situation or if there are other faults or deficiencies, contact your dealer for replacement parts or for service.

NB! The manufacturer responsibility for CE marking is invalidated if any form of modification is carried out.

Extra equipment

- Comfort discs for handles
- Gloves for handles
- Pedals
- Floor plates

Technical specifications

Weight	8,3 kg	Height:	45 cm	Width:	59,5 cm	Length:	56 cm
--------	--------	---------	-------	--------	---------	---------	-------

Recycling

This machine is comprised mainly of steel and plastic. When separating for recycling, the handles and plastic details must be handled as hazardous waste. The remaining parts of the machine can be handled as metal.

The "CE" mark indicates that this product conforms with European standards for health, safety and the environment.

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